THIS ARTWORK IS PROPERTY OF:

SIFI

PRODUCT NAME:

INS Klimi Med

PRODUCT CODE:

1G804690T1

COUNTRY:

ITA - FRA - UK - SPA1st Draft: 28/11/2018SEPARATIONS: (**Black - Fustella**)FORMULAZIONE: **Salviette**RICHIESTA DMS n°: **341**MEASURES (mm): **150x420**FOLDER (mm): **150x42**

CUTTING PROFILE:

FONT SIZE: **8 pt**SPACE: **9 pt****FRONTE**

ISSUE DATE

07-12-18

OPERATOR

G. Longo

DRAFT

03**TEXT BRAILLE:**

Klimi MED

IT

**Salviette monouso sterili per l'igiene perioculare
a base di acido ialuronico e clorexidina diglucosato**

Indicazioni

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Klimi MED

EN

Sterile disposable wipes for periocular hygiene containing hyaluronic acid and chlorhexidine digluconate

Indications

KLIMI MED is a soft disposable sterile wipe containing hyaluronic acid which provides detergent, emollient, soothing and decongestant activities.

The solution which keeps the wipe moist is chlorhexidine gluconate, an anti bacterial agent.

KLIMI MED is a medical device indicated for the hygiene and restoration of the physiological conditions of the eye and its appendages, in cases of infectious and inflammatory processes particularly before and after ophthalmic surgery, in combination with adequate medical therapy.

KLIMI MED is parabens and perfumes free.

Composition

Sodium hyaluronate, ammonium glycyrrhizinate, chlorhexidine digluconate, disodium phosphate dodecahydrate, sodium phosphate monobasic monohydrate, disodium edetate, polysorbate 20, glycerin, sodium hydroxymethylglycinate, citrus aurantium amara flower distillate, water.

Presentation

1 box containing 14 disposable sterile wipes.

1 leaflet.

Instructions for use

Wash your hands thoroughly and open the sachet to remove the moist wipe.

Close the eye, place the wipe over the eyelid and lashes and gently massage until the entire eye area is thoroughly cleansed.

In case of persistent secretions you can heat in warm water the closed sachet for a few minutes and use the wipe as a warm compress and massage gently.

In the case of inflammation it is possible to cool the closed sachet in a refrigerator for a few hours and place the open wipe on the eye closed for a few minutes as a cold compress.

Precautions and warnings

 Do not use **KLIMI MED** if the container is opened or damaged because the functionality and safety of the product is guaranteed by the integrity of original packaging.

KLIMI MED is a disposable sterile wipe, therefore it is intended to be used once only to avoid unintentional contamination of the periocular surface.

It is important to always use a separate wipe for each eye in order to avoid possible cross-contamination.

KLIMI MED contains chlorhexidine digluconate and therefore it is not recommended for use in neonates. The product should not be used by persons allergic to formaldehyde because it contains sodium hydroxymethylglycinate.

Occasionally this product may cause mild and transient burning sensation or mild local irritation.

If any of the above mentioned reactions occur, discontinue the use of this product.

Do not dispose of the product after use.

Keep away from dust and store in a cool, dry place, at temperature not above 25 °C.

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The use of **KLIMI MED** by children or persons with limited abilities should be supervised by a responsible adult.

Contraindications

Hypersensitivity to the product or to any of its components.

Interactions with drugs or other medical devices for ophthalmic use

There are no known interactions of the product with the administration of drugs or other medical devices for ophthalmic use.

READ CAREFULLY THE INSTRUCTIONS BEFORE USE OF PRODUCT

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SIFI S.p.A. – Via Ercole Patti, 36 – 95025